

HOW TO USE



Tear along one side of the foil, removing the condom carefully. Condoms are strong but can be torn by sharp finger nails or jewellery.



Only put the condom on, when the penis is erect and before contact with your partner's body. This helps prevent STD and pregnancy.



Place the condom over the end of the penis with the roll on the outside. With one hand, pinch the teat of the condom to expel any trapped air. This will make space for the semen.



Using the other hand, roll the condom down the length of the penis to its base.



Withdraw the penis soon after ejaculation whilst still erect, holding the condom firmly in place at the base of the penis. Wait until the penis is completely withdrawn before removing the condom. Keep the penis and the condom away from the vagina, to avoid any contact with semen. After use, wrap the condom in a tissue and dispose off hygienically.



Hot stuff at www.kamasutra.in

REMEMBER

- Always check the expiry date on the pack before purchase.
- If the date has expired, or if the foil is damaged do not use the condom
- Do not use oil-based lubricants such as petroleum jelly, mineral oil, vegetable oil, massage oil, butter margarine or cold cream as these could damage the condom, if you want additional lubricant, use only water based lubricants.
- Do consult a doctor or pharmacist about the compatibility with condoms or over-the-counter topical medicines that are applied to the penis or the vagina.
- No condom provides 100% protection; but when properly used, KamaSutra condoms help prevent pregnancy and can reduce the risk of Sexually Transmitted Diseases including HiV/AIDS.
- Never use a condom more than once.
- Store in a cool dry place away from heat and direct sunlight.
- This product contains natural rubber latex, which may cause allergic reactions in some individuals.

For **longlast** **MUTUAL ORGASMAX** only

- This product has an active ingredient Benzocaine (4.5%) and is not to be used for Oral sex.
- Warnings: Premature ejaculation may be due to a condition requiring medical supervision. If this product, used as directed, does not provide relief, discontinue use and consult a doctor. Avoid contact with the eyes. If you or your partner develop a rash or irritation, such as burning or itching, discontinue use. If symptoms persist, consult a doctor. Wash off lubricant after intercourse.

Marketed by: Godrej Consumer Products Ltd. Registered Address: 4th Floor, Godrej One, Pirojshanagar, Eastern Express Highway, Vikhroli East, Mumbai-400079.

Manufactured by: Raymond Consumer Care Ltd.

G- 35 & 36, MIDC, Waluj, Aurangabad, Maharashtra - 431136, India

M.L.MFG/MD/2020/000239

Contact: Manager (Consumer Relations), Godrej Consumer Products Ltd.,

Contact at the above registered address. Toll Free No. 1-800-266-0007. E-mail: care@godrejcp.com

® Registered trademark and copyright of Godrej Consumer Products Ltd.