PRODUKTNAME: KLY® GLEITGEL

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CODE: 150.06.05.0690 Rev:09/24.11.2021

KLY® LUBRICATING JELLY

PRODUCT NAME: KLY® LUBRICATING JELLY

COMPOSITION: Deionized water, Natural Glycerin, MonopropylenGlycol, Hydroxy Ethyl Cellulose, Methyl Hydroxybenzoate, Citric Acid

INTENDED USE: This gel is used in endoscopy, colonoscopy, gynecological examinations, rectal endoscopic, catheter operations, also used for vaginal lubrication.

INSTRUCTION FOR USE: To Open The Tube: Remove and reverse the cap then puncture the tube with it. Apply Lubricating Jelly to the gloves or the instrument in sufficient amount so that it forms a film over the area that comes in contact with patient. Wash off after use.

To Open The Sachet: Open carefully from marked area.

SPECIAL PRECAUTIONS: Precautions: If container is damaged or opened, the Product is not guaranteed.

CONTRAINDICATION: There is no known contraindication

OVERDOSAGE AND SIDE EFFECTS: There is no known symptoms for overdose usage.

SENSITIVITY TO GLYCERIN: Glycerin has emollient properties. Precautions Shall be taken when using the gel in people with sensitivity to glycerin.

IDENTIFICATION: Transparent, water-soluble, greaseless and jelly.

PRESENTATION: Tubes Containing either: 42 g, 50 g, 82 g, 113,4 g, 100 g or 250 g

SACHETS CONTAINING EITHER: 2,5 g, 3 g, 3,5 g, 4 g, 4,5 g, 5 g, 10 g

STORAGE INSTRUCTIONS: Store between 5-30°C

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CE
2292

Made in
Turkey
Discover
the potential

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